


Myobrace® for Kids - Broad

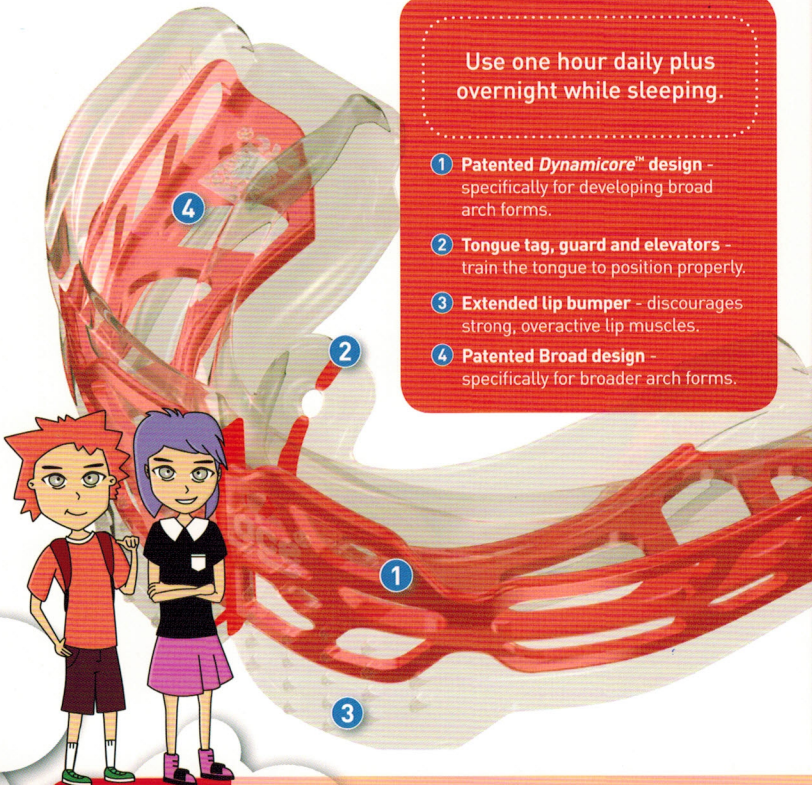

Use one hour daily plus overnight while sleeping.

- 1 Patented *Dynamicore™* design - specifically for developing broad arch forms.
- 2 Tongue tag, guard and elevators - train the tongue to position properly.
- 3 Extended lip bumper - discourages strong, overactive lip muscles.
- 4 Patented *Broad design* - specifically for broader arch forms.

mixed Dentition

The Myobrace® for Kids – Broad appliance series is a three-stage system with a specialised arch form for specific genetic and racial groups. This wider arch form is prevalent in many Asian countries, as well as Central and Latin American groups. The appliance is characterised by its flatter anterior arch form with wider canine area, and a more flared posterior arch form. Ultimately, the natural arch form is determined by the tongue position and the *K Broad* series treats breathing, corrects tongue positioning and arch form.



Designed for broad arch forms with:

- Class II Division 1 + 2 malocclusion.
- Class III.
- Anterior (upper + lower) crowding.
- Deep bite.
- Open bite.

6 - 12 YEARS

All Myobrace® appliances are designed to correct habits, develop the maxilla and mandible, and align the teeth. Each stage focuses on a particular treatment goal.



HABIT CORRECTION

Establish nasal breathing.

STAGE 1



4-6 months



The *K1 BROAD* is available in medium and large and comes in clear and yellow.

K1 BROAD APPLIANCE - PERSPECTIVE VIEW (left)

K1 BROAD - CROSS SECTION (above)

MYOBACE® K1 BROAD

The *K1 Broad* focuses on establishing nasal breathing and myofunctional habit correction.

It is soft and flexible which gives the best compliance, adapts to any arch form and malocclusion, and optimises staying in place at night.


Only move to the *K2 Broad* when the *K1 Broad* stays in at night and nasal breathing is established.




ARCH DEVELOPMENT

Establish correct tongue position.

STAGE 2

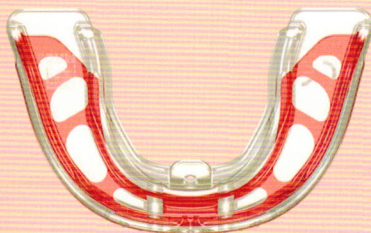

4-6 months

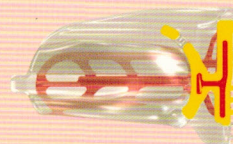

The *K2 BROAD* is available in medium and comes in clear and yellow with a red *Dynamicore™*.

K2 BROAD APPLIANCE - TOP VIEW (left)

K2 BROAD - CROSS SECTION (above)

MYOBACE® K2 BROAD

The *K2 Broad* focuses on arch development and continuing habit correction. The *K2 Broad* features a *Dynamicore™* which assists in developing the upper and lower arch form. This allows more space to establish correct tongue position and correct swallowing. This, in turn, improves dental alignment. Move to *K3 Broad* when arch form is improved, as well as tongue resting position and correct swallowing patterns are established.




FINAL ALIGNMENT AND RETENTION

Maintain correct lip posture and swallow.

STAGE 3



4-6 months

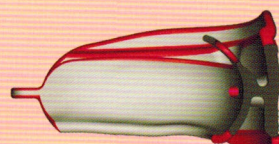

The *K3 BROAD* is available in medium and comes in red.

K3 BROAD APPLIANCE - REAR TECHNICAL VIEW (left)

K3 BROAD - CROSS SECTION (above)

MYOBACE® K3 BROAD

The *K3 Broad* focuses on completing habit correction, final tooth alignment and retention with its firmer polyurethane construction.

The hollow tongue tag facilitates final tongue position directly on the correct spot.